

# My Personal Information

Please provide the following information and bring this brochure with you to your next appointment:

My Apple device is:

---

My Apple ID is:

Username:

---

Password

---

I currently have the following Bluetooth devices paired with my Apple device:

---

---

---

---

---

---

Halo 2 and TruLink are compatible with iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Air, iPad (4th generation), iPad mini 3, iPad mini with Retina display, iPad mini, iPod touch (5th generation) and Apple Watch. Use of Apple Watch requires iPhone 5 or later. "Made for iPod," "Made for iPhone," and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPod, iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPod, iPhone, iPad or Apple Watch may affect wireless performance. Apple, the Apple logo, iPhone, iPad, iPod touch, iTunes, Apple Watch and App Store are trademarks of Apple Inc., registered in the U.S. and other countries.



[Starkey.com](http://Starkey.com)

# TRULINK™

Preparing for your  
Made for iPhone®  
Hearing Aid



## Congratulations!

This is an exciting time for you and your loved ones as you pursue better hearing. Your hearing care professional has placed the order for your Made for iPhone Hearing Aids. Because of the unique way that these hearing aids interface with Apple® devices, there are some actions needed **before your next appointment.**



Halo 2

Please complete the following three steps:

### STEP 1: Is your Apple device compatible with your Made for iPhone Hearing Aids?


Visit [TruLinkHearing.com/FAQ](http://TruLinkHearing.com/FAQ) for a full listing of Apple devices that are compatible with Made for iPhone hearing aids. Verify your device is compatible.

**If your current Apple device is not on this list, consider upgrading your device to take advantage of your hearing aids.**

### STEP 2: Does your Apple "device have the latest version\* of iOS software?

1. Check the iOS software on your Apple device to ensure the latest version\* is installed.

Tap **Settings > General > Software Update.**

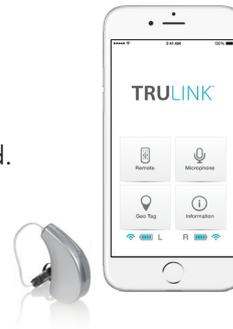

\*Must be 7.1 or higher.

2. If a software update is available, tap **Install.**

**NOTE:** You must be connected to WiFi to update iOS.

### STEP 3: Have you downloaded the TruLink Hearing Control app?

1. Tap the **App Store® icon.**
2. Type **TruLink** in the search field.



3. Once the TruLink Hearing Control app screen is displayed, tap **Get** (it will change to **Install**).



4. Tap **Install** to load the app on your device.

**NOTE:** Although this is a free app, you will be prompted to sign into your iTunes® account.

5. Type your Apple ID and password to complete the download process.

**You have now completed all the steps necessary to prepare for your fitting appointment.**